## Easy Expression™ Bustier

Thank you for choosing Easy Expression Bustier!

### Why Easy Expression™ Bustier?

Easy Expression Bustier is the best solution, if you want to read a magazine, surf the Internet or play with your older child while pumping. The comfortable Easy Expression Bustier enables you to pump hands-free. Easy Expression Bustier is made of a soft cotton/elastane blend, is easy to wear and available in different sizes.

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#### Instructions

**1** Dress bustier, leave zipper a little bit open.

**2** Place breastshield under bustier so that your nipple is centred in the tunnel of the breastshield. Then close the zipper.

**3** Attach bottle with connector to the breastshield. Now pump handsfree.

For a smooth and secure pumping session with Easy Expression Bustier Medela recommends sitting upright.

#### Wash instructions

- I Wash with similar colour only.
- I Machine wash, max temperature 30 °C.
- I Only non-chlorine bleach when needed.
- I Tumble dry at lowest temperature only.
- I No iron.

#### Material

93 % Cotton, 7 % Spandex

Visit our website [www.medela.com](http://www.medela.com) there you can find a detailed description of the correct use of Easy Expression and more information about Medela products.

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